


Perfect Pupil® Injectable, milvella


Tricks to remember:

- Operate on a medium sized pupil first for first time training (4-5mm size)
- Incision size: phaco tip plus 0.4 mm
- Use visco-elastic to enlarge pupil temporarily and during surgery for iris protection
- Load the PerfectPupil into the Injector just prior to insertion in the anterior chamber



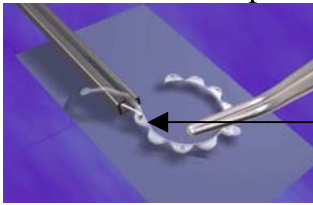


Place injector shaft underneath arm, engage hook into most proximal hole


- Insert tangentially to the pupil and on top of the iris

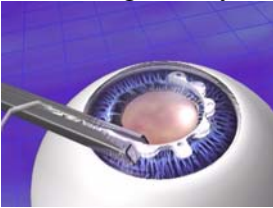


- Use a Sinsky hook to engage proximal tab first, then middle, and distal tab last

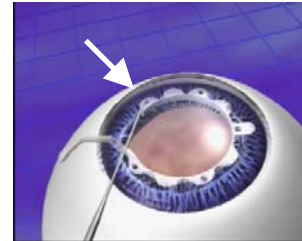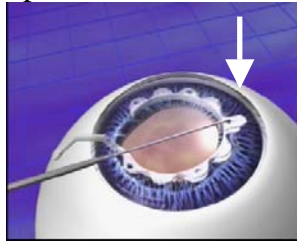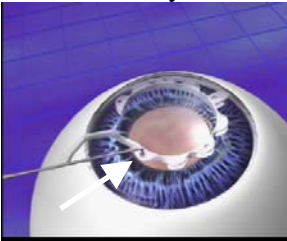

- Disengage Perfect Pupil® by pulling with a Sinsky hook in between the tabs in inverse order of engagement: first the distal, then the middle, lastly the proximal tab.

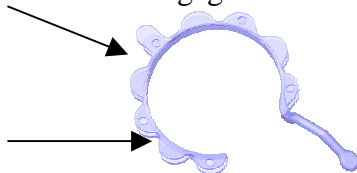

- Enlarge incision under the arm with a spatula, while lifting Perfect Pupil® upwards

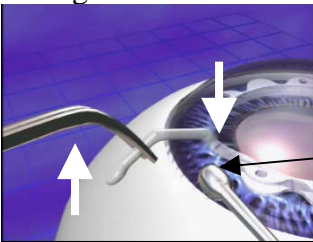

Push incision down with spatula. Lift up Perfect Pupil® with forceps

- Move Perfect Pupil® to the left when the middle tab slides through wound

