

## INSTRUCTIONS FOR USE

### Perfect Pupil®

#### Description

*Perfect Pupil* is a single use sterile pupil dilator used for enlarging a small pupil during cataract surgery. The device comprises a ring with U shaped cross sections for engaging the iris. There are holes used to position the device, and a distal tab to engage the pupil margin firmly to help prevent dislodgement. An arm remains external to the corneal incision to aid positioning and removal.

#### Material

Polyurethane.

#### Diameter

*Perfect Pupil* is available with an internal diameter of 7mm

#### Indications

*Perfect Pupil* is intended for enlarging a small pupil primarily to facilitate cataract surgery, but may be indicated for other forms of eye surgery where a large pupil is preferred.

#### Contraindications

Corneal endothelial pathology.

#### Sterilization

*Perfect Pupil* is sterilized by gamma-irradiation.

#### Storage

Store package window up in dry conditions. Store below 45°C.

#### Complications of use

Possible abrasion or tearing of the corneal endothelium, Descemet's membrane, or other intraocular structures.

#### Warnings

- DO NOT USE IF THE PACKAGE IS DAMAGED
- SINGLE USE ONLY - DO NOT RE-STERILIZE.

#### Precautions

This device should only be used by a surgeon trained in ophthalmic surgery. A surgeon should ensure that the use of this device is well understood before use.


#### Package Opening


Peel off the shrink wrap, open the polystyrene box and remove the sealed pouch. The contents of this pouch are sterile unless damaged or opened.


Under aseptic conditions, open the sealed pouch and drop the device package onto a sterile tray. Peel the label off this package and remove *Perfect Pupil* with forceps via the slot provided.

Under aseptic conditions, open the sealed pouch and drop the device package onto a sterile tray. Peel the label off this package and remove *Perfect Pupil* with forceps via the slot provided. *Perfect Pupil* should be washed with balanced salt solution prior to insertion.

 Sterilized by Radiation

 Do not re-use

 Use by (YYYY-MM)

 See Instructions for Use

Note

*Perfect Pupil* is a Trade Mark of Milvella Ltd of Australia, and is not intended to imply that the use of this device will ensure the pupil will be perfect.